



These 3 LAYER MASKS were made by the San Francisco Opera Costume Department

**THIS PACKAGE CONTAINS \_\_\_\_\_ MASKS**

**MATERIALS:**

**FRONT LAYER:** 100% Cotton **or** 50/50 Poly/Cotton Blend  
**MIDDLE LAYER:** Pellon 50 (100% Non-woven, Non-Fusible)  
**BACK LAYER (Face Contact Layer):** 100% Kona Cotton ( Non-printed layer )  
**FABRIC TIES:** 100% Cotton **or** 50/50 Poly/Cotton Blend  
**METAL MILLINERY WIRE (for Bridge of Nose)**

**FOR FIT:**

- 1)Mask should be placed on face with the wire at the top. Bend the wire to fit the bridge of the nose.
- 2)Take note of which side of the fabric is next to your face.

**WASH & CARE:**

**It is recommended the masks must be sterilized before use.**

Recommended wash is with high temperature with laundry detergent.  
And tumble dry on hot.

Masks should be washed after every use and/or as frequently as possible.

**WARNINGS:**

**These masks are not certified protective medical face mask for COVID-19 VIRUS**

- These face-masks are **NOT** Surgical Masks or **Filtering Face-Piece Respirators (FFRs)**.
- These face masks have not been tested for fluid resistance, particulate or bacteria filtration efficiency, differential pressure, or flammability.